


Replace the battery

1. Remove

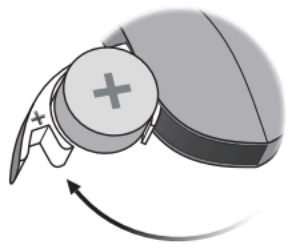

Fully open the battery drawer. Remove the battery.

2. Uncover



Remove the sticky label from the + side of the new battery.

Tip:
Wait 2 minutes so that the battery can draw air, to make sure to function optimally.

3. Insert



Insert the new battery into the battery drawer. Make sure the + side is facing up.


4. Close



Close the battery drawer. The hearing aid will play a jingle through the earpiece.

Hold the earpiece close to your ear to hear the jingle.

Tip

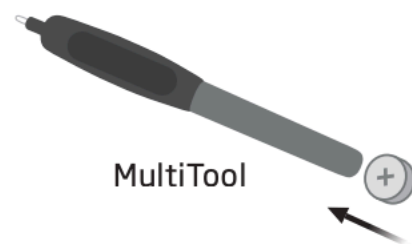

The MultiTool can be used to change the battery. Use the magnetic end to remove and insert batteries.

The MultiTool is provided by your hearing care professional.